


The UAM Ventilator PRE-OPERATIVE CHECKLIST



I. Perform at the START of every OPERATING session

INSPECT & TEST

1. Perform UAM pre-operative checks according to its laminated checklist
2. Ensure that all tubing and connections are secure as in the illustration:

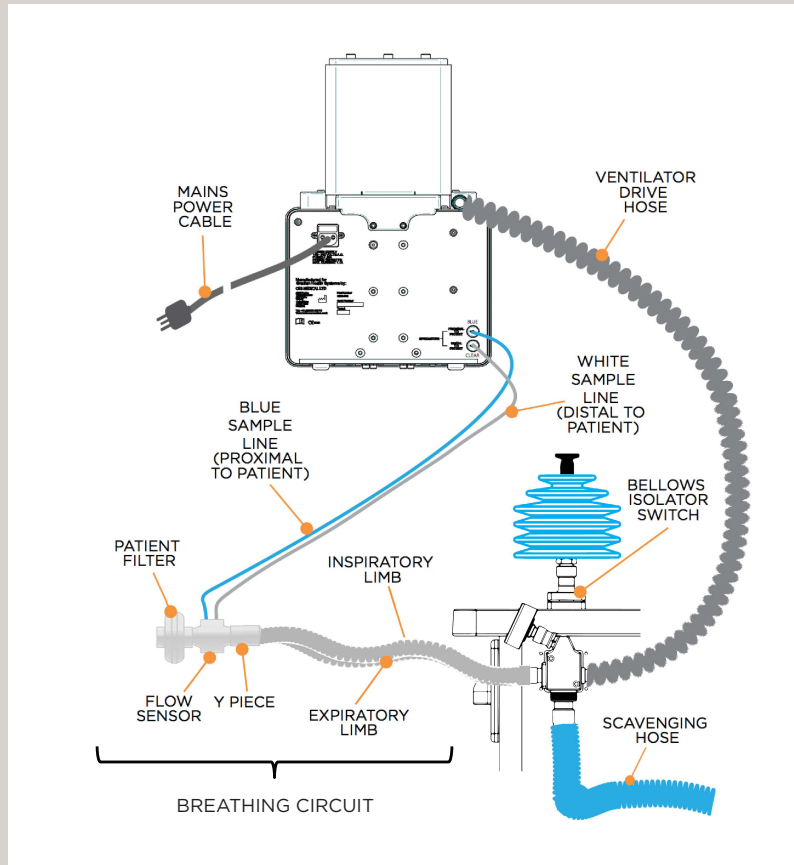

3. Inspect the flow sensor and ensure that the blue line is on the patient side
4. Ensure that the breathing circuit is not connected to the patient at this point

POWER

1. Ensure that the ventilator's power cord is plugged into one of the UAM's receptacles
2. Turn on the main power switch on the back of the UAM
3. Check that the light by the ventilator power switch is yellow. This means that the Ventilator is connected to the electrical supply and that its batteries are charging
4. Press the power button on the front of the ventilator. The light by the button should turn green and the ventilator will power up and perform its start-up sequence. The ventilator bellows should move up and down during start-up

Note: It is recommended to leave the ventilator powered on for the entire duration of the operating session. This will avoid having the ventilator perform its start up sequence before each case.



The UAM Ventilator PRE-OPERATIVE CHECKLIST

II. Perform BEFORE each case

1. Ensure that the flow sensor is not cracked and is free of condensation, debris and fluids. If it is damaged or contaminated replace it with a new one and perform a flow sensor calibration
2. Place a new bacteria filter on the flow sensor

III. Perform at the END of each case

1. Dispose of the used bacteria filter
2. Ensure that the flow sensor is not cracked and is free of condensation, debris and fluids. If damaged or contaminated replace and perform flow sensor calibration
2. Clean any surfaces that may have been contaminated or soiled

Note: It is recommended to leave the ventilator powered on for the entire duration of the operating session. This will avoid having the ventilator perform its start up sequence before each case.

IV. Perform at the END of every OPERATING session

1. Turn off the ventilator
2. Check that the main power switch on the back of the UAM is in the ON position to charge the ventilator batteries
3. Wipe down the ventilator with a cloth dampened with a mild detergent

V. Perform EVERY TWO WEEKS

1. Clean the flow sensor per the instructions in the ventilator User Guide
2. Calibrate the flow sensor per the instructions in the ventilator User Guide

For service issues, please contact
Gradian Health Systems at service@gradianhealth.org

