

GRADIAN CCV

USING THE BATTERIES

The Gradian CCV has both *internal* and *external* battery backups. To ensure maximum backup charge, the *external* battery should **only be used** during a prolonged mains outage or during inter-facility transport.



While mains power is available:

Keep the external battery disconnected from the CCV



If mains power is lost:

Connect the external battery to the CCV



When mains power returns:

Disconnect the external battery from the CCV




GRADIAN CCV

CHARGING THE BATTERIES

The Gradian CCV has *internal* and *external* battery backups. Both should always be left on charge—even **when not in use**—to ensure that maximum backup is available if needed.

Ensure that at all times:

- The CCV is connected to the power extender
- The external battery is connected to the power extender
- The power extender is connected to mains

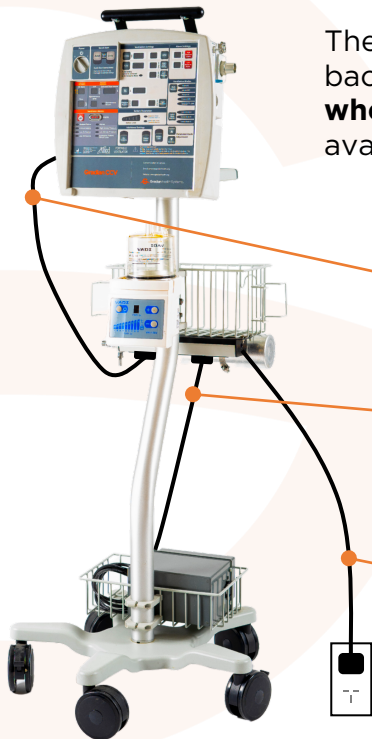

For more details, see:
<https://youtu.be/wuoAzvAWYVO>